

Crowns/Bridges

Inlays/Onlays

Transfer/Implant Impression

Functional Impression

Flexitime®

Confidently flexible
and precise!

Go
double-speed:
Twice as fast
as before!

High-speed precision with Flexitime and the Dynamix speed system!

Practical cartridges and the new functional Dynamix speed mixer are perfectly matched in this well thought-out system featuring all the advantages of automatic mixing for impression materials

- * High-speed mixing and precise dispensing
- * Homogeneously and void-free mixed impression materials
- * 3 selectable velocities



Cartridges marked with Dynamix speed Symbol *

With the Flexitime Dynamix speed system, dentists gain

* Efficient and easy handling

The new device provides even more time savings, due to double-speed automatic mixing and is easy and intuitive to use. Its powerful propulsion and advanced handling properties provide straightforward and reliable employment.


* Highly reliable material quality

Automatic mixing guarantees homogeneously mixed impression material in consistent, void-free quality for precisely fitting restorations – independent of the operator.

* Superior process safety

No propulsion of pistons with open cover – prevents the risk of material splashes and injuries. Filling level indicator indicates cartridge change.

Twice as fast, even with Putty material!

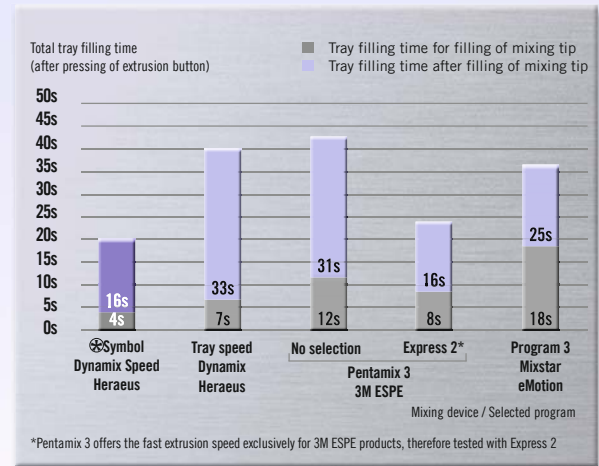

Tray filling time Flexitime Dynamix Putty, average tray size (40 ml)

Source: Heraeus Research & Development, data on file.

Speed selection with Dynamix speed

Selecting tray velocities

- * After insertion of a cartridge suited for **high-speed** extrusion (marked with Dynamix speed symbol Ⓢ), press according speed selection button for **double speed**.
- * After insertion of a cartridge without speed symbols, you do not have to select a speed. Standard speed (ST) will automatically be set.
- * The LED-light is confirming your choice/selected speed.

- * Selected speed stays active until cartridge replacement or if you actively switch to standard speed.
- * Close lid and press button for tray filling to start extrusion.

Selecting syringe velocity

- * After insertion of cartridge, close lid and press button for syringe filling with adapted speed to start extrusion.

Product

Order no.

Refills

- * Flexitime **Dynamix Putty Refill** _____ **66035991**
3 x (2 x 380 ml) Dynamix Putty, 3 x 2 Dynamix Mixing Tips,
3 x 1 Dynamix Fixation Ring
- * Flexitime **Fast & Scan Dynamix Putty Refill** _____ **66044882**
3 x (2 x 380 ml) Fast & Scan Dynamix Putty,
3 x 2 Dynamix Mixing Tips, 3 x 1 Dynamix Fixation Ring
- * Flexitime **Dynamix Heavy Tray Refill** _____ **66035992**
3 x (2 x 380 ml) Dynamix Heavy Tray,
3 x 2 Dynamix Mixing Tips, 3 x 1 Dynamix Fixation Ring
- * Flexitime **Fast & Scan Dynamix Heavy Tray Refill** _____ **66044755**
3 x (2 x 380 ml) Fast & Scan Dynamix Heavy Tray,
3 x 2 Dynamix Mixing Tips, 3 x 1 Dynamix Fixation Ring
- * Flexitime **Dynamix Monophase Refill** _____ **66035993**
3 x (2 x 380 ml) Dynamix Monophase,
3 x 2 Dynamix Mixing Tips, 3 x 1 Dynamix Fixation Ring

Product

Order no.

Refills

- * Flexitime **Medium Flow Refill** _____ **66041062**
6 x (2 x 50 ml) Medium Flow, 6 x 12 Mixing Tips yellow
- * Flexitime **Fast & Scan Medium Flow Refill** _____ **66045006**
6 x (2 x 50 ml) Fast & Scan Medium Flow,
6 x 12 Mixing Tips yellow
- * Flexitime **Correct Flow Refill** _____ **66002186**
6 x (2 x 50 ml) Correct Flow, 6 x 12 Mixing Tips yellow
- * Flexitime **Light Flow Refill** _____ **66041061**
6 x (2 x 50 ml) Light Flow, 6 x 12 Mixing Tips yellow
- * Flexitime **Fast & Scan Light Flow Refill** _____ **66045009**
6 x (2 x 50 ml) Fast & Scan Light Flow,
6 x 12 Mixing Tips yellow

