

# Heraeus

Crowns/Bridges

Inlays/Onlays

Transfer & Implant Impressions

Functional Impressions

## Variotime®

### Freedom to impress

Go  
double-speed:  
Twice as fast  
as before!

#### High-speed precision with Variotime and the Dynamix speed system!

Practical cartridges and the new functional Dynamix speed mixer are perfectly matched in this well thought-out system featuring all the advantages of automatic mixing for impression materials.

- High-speed mixing and precise dispensing
- Homogeneously and void-free mixed impression materials
- 3 selectable velocities



Cartridges marked with Dynamix speed Symbol \*

[www.heraeus-dental.com](http://www.heraeus-dental.com)

**Variotime®**  
The right impression. Everytime.

## With the Variotime Dynamix speed system, dentists gain

### ■ Efficient and easy handling

The new device provides even more time savings, due to double-speed automatic mixing and is easy and intuitive to use. Its powerful propulsion and advanced handling properties provide straightforward and reliable employment


### ■ Highly reliable material quality

Automatic mixing guarantees homogeneously mixed impression material in consistent, void-free quality for precisely fitting restorations – independent of the operator.

### ■ Superior process safety

No propulsion of pistons with open cover – prevents the risk of material splashes and injuries. Filling level indicator indicates cartridge change.

## Twice as fast, even with Putty material!

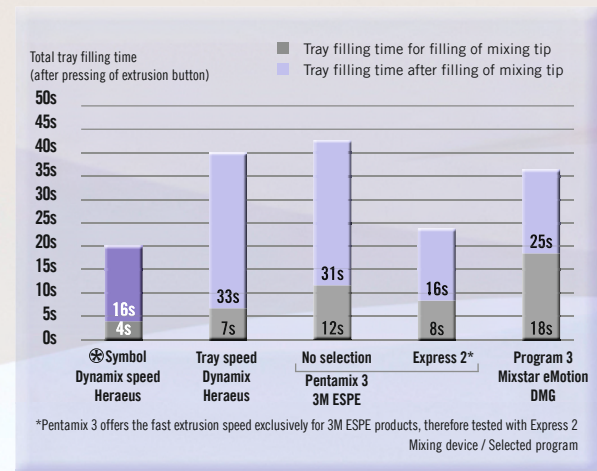

Tray filling time Variotime Dynamix Putty, average tray size (40 ml)  
Source: Heraeus Research & Development, data on file.

## Speed selection with Dynamix speed

### Selecting tray velocities

- After insertion of a cartridge suited for **high-speed** extrusion (marked with Dynamix speed symbol ⊕), press according speed selection button for **double speed**.
- After insertion of a cartridge without speed symbols, you do not have to select a speed. Standard speed (ST) will automatically be set.
- The LED-light is confirming your choice/selected speed.

- Selected speed stays active until cartridge replacement or if you actively switch to standard speed.
- Close lid and press button for tray filling to start extrusion.

### Selecting syringe velocity

- After insertion of cartridge, close lid and press button for syringe filling with adapted speed to start extrusion.

## Product

## Order no.

### Refills

- Variotime Dynamix **Putty Refill** \_\_\_\_\_ **66044995**  
3 x (2 x 380 ml) Dynamix Putty,  
3 x 2 Dynamix Mixing Tips, 3 x 1 Dynamix Fixation Ring
- Variotime Dynamix **Heavy Tray Refill** \_\_\_\_\_ **66045000**  
3 x (2 x 380 ml) Dynamix Heavy Tray,  
3 x 2 Dynamix Mixing Tips, 3 x 1 Dynamix Fixation Ring
- Variotime Dynamix **Monophase Refill** \_\_\_\_\_ **66045034**  
3 x (2 x 380 ml) Dynamix Heavy Tray,  
3 x 2 Dynamix Mixing Tips, 3 x 1 Dynamix Fixation Ring

## Product

## Order no.

### Refills

- Variotime **Medium Flow Refill** \_\_\_\_\_ **66045036**  
6 x (2 x 50 ml) Medium Flow, 6 x 10 Mixing Tips yellow
- Variotime **Light Flow Refill** \_\_\_\_\_ **66045038**  
6 x (2 x 50 ml) Light Flow, 6 x 10 Mixing Tips yellow
- Variotime **Extra Light Flow Refill** \_\_\_\_\_ **66045040**  
6 x (2 x 50 ml) Extra Light Flow, 6 x 10 Mixing Tips yellow

